


HOW TO


Sign up and setup insulin calculator

Sign up


1 Create Hedia account and set up insulin calculation settings

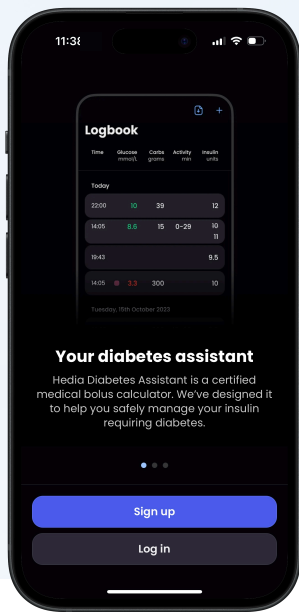


Choose if you are signing up as a new user or logging in to an already existing account.

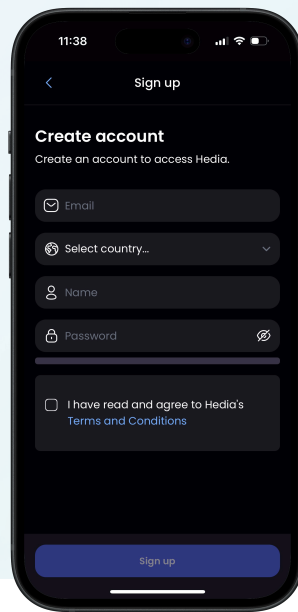


Enter name, country, email and a password and accept terms and conditions.

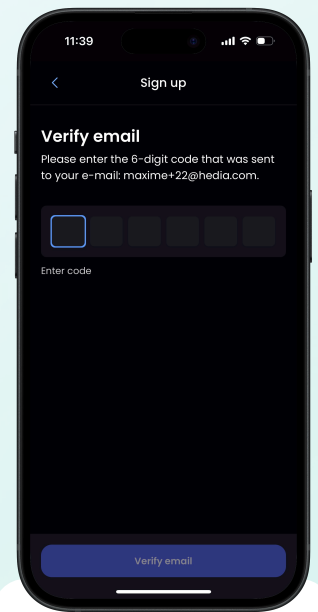

Enter the 6-digit code you received by email.

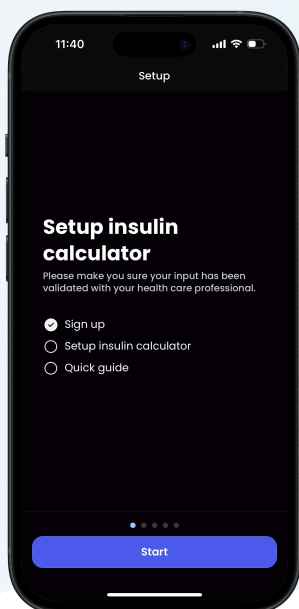


Sign-up is done.

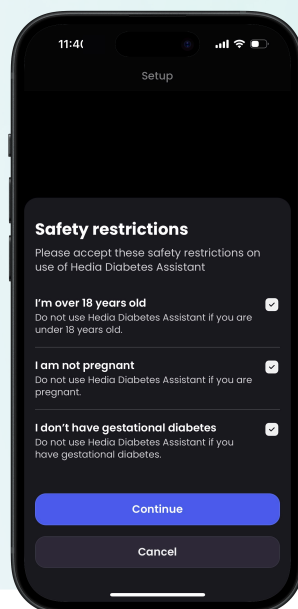


Accept the safety restrictions.


Setup insulin calculator

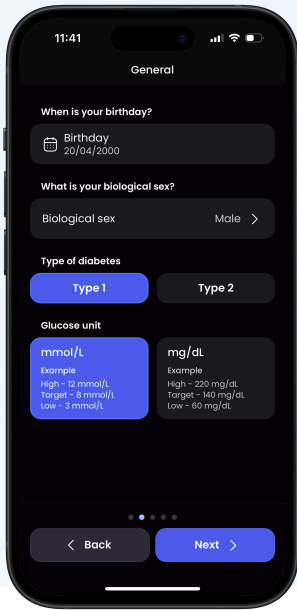

Enter birthday, biological sex, diabetes type and preferred glucose unit (mmol/L or mg/dL).

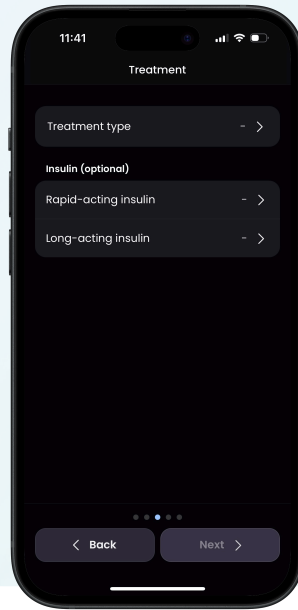


Enter treatment type: Pen (half or whole unit) or Pump.

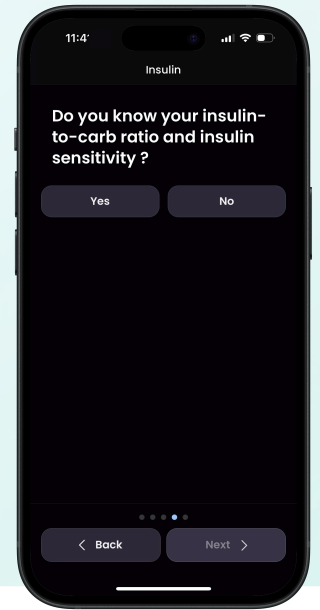


Do you know your Insulin-to-carb ratio and insulin sensitivity?


YES NO

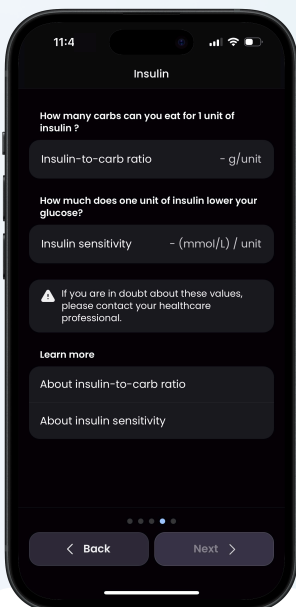


Enter Insulin-to-carb ratio and insulin sensitivity.

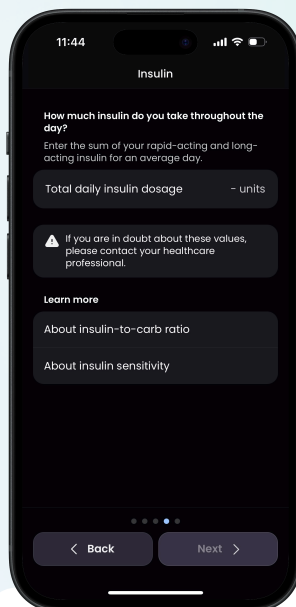

Enter you Total daily insulin dosage.

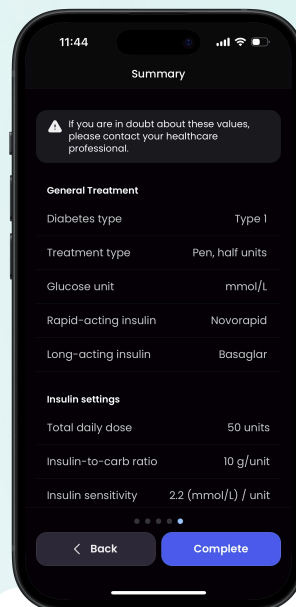

Check your summary.

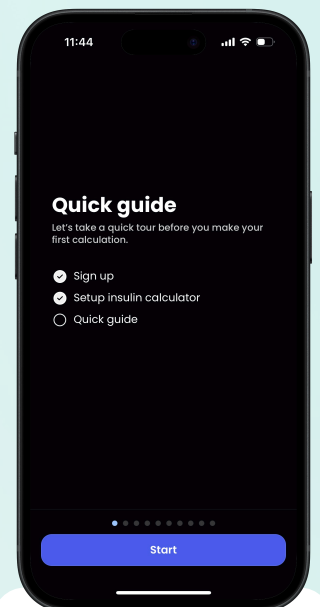

Setup is done.

2 Go through Quick guide

